

## Precautions

1. Always exercise care when operating and handling.
2. Do not disassemble any portion of the charger and adapter. It will void any product warranty.
3. Do not use any power adapter other than the one provided with the device or acquired from the manufacturer or its partners.
4. In the unlikely event that abnormal noise, strange odor, or smoke is present, immediately disconnect all power sources. Please report the problem to your device provider immediately.

## Specifications

I/O Ports	
DC-in	1
Charging Slot	4

Mechanical and Environmental	
AC/DC Adapters	Input: 100 – 240V AC; Output: 19V DC, 10.52A
Dimension	23.2cm(W) x 15.7cm(D) x 8.5cm(H)
Regulation	FCC Class B, CE, C-Tick, RoHS compliant
Temperature	Operation: 0°C – 40°C; Storage: -20°C – 60°C
Humidity	0% – 90% non-condensing

## GC590

### BASIC OPERATION GUIDE



### ENGLISH

#### INTRODUCTION

GC590 4-Bay Gang Charger Kit (ACC-001-590M) is available for charging the DR202 batteries of DT590, DT592 and DT594 series Medical Cart Computers. The Kit includes one battery cradle, an AC-DC adapter and a power cord.

The battery cradle features a DC-input jack, four battery slots and four LEDs for power and charging indicators. Connect the AC-DC adapter cable to the DC-input jack and then insert the battery packs one by one for charging.

After inserting the battery pack in the slot, the charging LED lights up (orange) if the battery pack is being recharged. When charging is complete or if the battery pack is already fully charged, the LED will show solid blue.

#### LED Indicator Status

- Power On/ Off: LED off
- Battery Fully Charged: Blue on
- Charging Battery: Orange on

#### Package Content

1. One GC590 4-Bay Gang Charger
2. One AC-DC adapter with power cord

